



Cleaning Guide

WOW-001D-XXX



\* If in doubt regarding this step please remove battery from service and contact technical support.

### Important Information - Installation

It is recommended that the charging of any batteries should not be undertaken within a patient care environment and preferably in a room that can be closed to isolate any smoke and fumes from other areas.  
It is also recommended that combustible materials should be kept away from the vicinity of any chargers.

If in doubt please consult the GeniTec Charger User Manual (IFU) for addition important safety recommendations and information.  
(see QR codes to the left)

GeniTec Charger  
User Manual

Use QR code Below:



GeniTec Charger  
Cleaning Guide

Use QR code Below:





### Cleaning Guide

WOW-001D-XXX

**!** \* If in doubt regarding this step please remove battery from service and contact technical support.




**1** Remove Power module

Press the release button located at the top of the dock .



**2\*** Inspect for cracks or damage.

- Main seal around battery.
- Blue bungs are present.
- LED bargraph panel inset is intact.
- Service LED is not illuminated

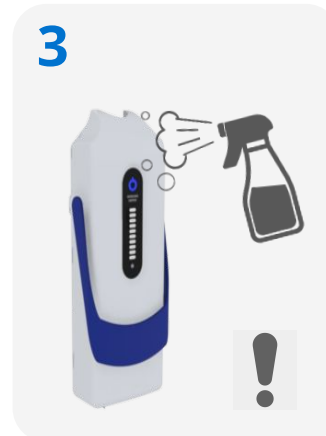

**3** Clean battery module with:

- Hot soapy water
- Hospital grade sanitisers
- IPA Disinfectant
- Alcohol based solution



**4** Dry battery module with:

- Cleaning Wipes
- Antibacterial wipes
- Dry Cloth